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MODEL :
DATE :
DEALER :

Observations:

1. This product is covered by a 6-month warranty
2. During the period of time, we shall provide services free of charge for all failure derived from non-human factors by presenting this card.
3. All repairs of units not dismantled by us, a service charge shall be due.
4. This card is valid without seal of the dealer.

Use Instruction Manual

Pending patent

Features

1. The needle hole is visible and applicable to a number of eyebrow streakers.
2. The replaceable needle holding tube is removable for cleaning and disinfection, it is also disposable for replacements and is free from loosening and dropping.
3. The product in reference comes with a variable-speed switch on top of the device providing a handy and prompt adjustment of speed.
4. In order to keep the white colorant from back-flow into the needle hole and the needle holding tube (with the needle hole, the white colorant will generate back-flow and there is no way to stop it); the most effective prevention is to provide with a sponge to keep the colorant from back-flowing or another colorant to remove the back-flow for life span of the device.

Features provided to fight the flaws in the eyebrow streakers available on the market:

- (1) Detachable eyebrow streaker housing, protective tube and unit cover
 - (2) The needle holding tube, needle hole are accessible for cleaning and removal for periodical maintenance so as to extend the use of the device.
5. The vertical jump of the needle assures a stable operation and enhanced friction resistance as well as speeding handling as a compact and durable unit.
 6. By pushing up or pulling down the protective tube, the needle is duly adjusted as a handy needle sleeve adjustment feature to insert the needle into skin to a fixed depth.
 7. All products are subject to sterilization ruled by sanitary requirements and are packed in units bearing disinfection instructions, please dispose it after use.

Observations

1. Please read carefully the Operation Instruction before use or check with the sales people.
2. To keep out infection by bacteria and safety of the customer, you are kindly requested to use brand new eyebrow streaking needles with casing and needle holding tube as well as sponge or Needle Sleeve.
3. After operation, the used eyebrow streaking needle, needle cover tube and sponge shall be removed and disposed accordingly.
4. After use, please check the needle holding tube, in case of remaining pigment between the needle holding tube and the sleeve, please remove the sleeve for cleaning or disinfection, then leave a drop of exclusive lubricant on the needle holding tube, then insert the interior needle into the needle holder and finally press it slightly into the housing and return to the original position.
5. Please do not leave the eyebrow streaker subject to heavy impacts.